


— USAGE RECOMMENDATIONS —

	MODE 1 ANTI-AGEING	MODE 2 PURIFYING	MODE 3 PIGMENTATION	MODE 4 POST INJECTABLE
INTENSIVE COURSE	<p>12 Treatments, 2 per week for first 4 weeks. Followed by 1 treatment per week for 4 weeks.</p> <p>Treatment duration: 20 minutes</p>	<p>12 Treatments, 2 per week for first 4 weeks. Followed by 1 treatment per week for 4 weeks.</p> <p>Treatment duration: 20 minutes</p>	<p>12 Treatments, 2 per week for first 4 weeks. Followed by 1 treatment per week for 4 weeks.</p> <p>Treatment duration: 20 minutes</p>	<p>Immediately post treatment for superior results. Complimenting skin needling & injectables.</p> <p>Treatment duration: 20 minutes</p>
REGULAR USAGE	<p>Weekly, fortnightly, monthly, or as required.</p> <p>Treatment duration: 10-20 minutes</p>	<p>Weekly, fortnightly, monthly, or as required.</p> <p>Treatment duration: 10-20 minutes</p>	<p>Weekly, fortnightly, monthly, or as required.</p> <p>Treatment duration: 10-20 minutes</p>	<p>Not to be used after:</p> <ul style="list-style-type: none"> • heat based treatments due to the integration of near infrared. • chemical peels • laser treatments • IPL
	MODE 5 CALMING	MODE 6 ANTI-AGEING EXPRESS	NEAR INFRARED	
INTENSIVE COURSE	<p>12 Treatments, 2 per week for first 4 weeks. Followed by 1 treatment per week for 4 weeks.</p> <p>Treatment duration: 20 minutes</p>	<p>12 Treatments, 2 per week for first 4 weeks. Followed by 1 treatment per week for 4 weeks.</p> <p>Treatment duration: 10 minutes</p>	<p>To add NIR to treatment press </p> <p>Can be combined with modes 1,2,3,5 & 6</p>	
REGULAR USAGE	<p>Weekly, fortnightly, monthly, or as required.</p> <p>Treatment duration: 10-20 minutes</p>	<p>Weekly, fortnightly, monthly, or as required.</p> <p>Treatment duration: 10 minutes</p>	<p>Weekly, fortnightly, monthly, or as required.</p> <p>Not to used after heat based treatments</p>	